# MAJOR AF COMP

The MAJOR AF COMP is a veneering system made of a line of composite multi-layer facings and teeth complemented with a range of light curing materials specifically designed to obtain life-like restorations on implant prosthesis.

The MAJOR AF COMP may be used for a wide range of dental treatments, both by the dentists in their own clinics and

by technicians in the laboratories. The multi-layered teeth and facings, in composite materials, have been developed to match natural aesthetics with functional concepts.

They may be used in combination with the MAJOR AF COMP of cements, bonding agents, light curing composites and acrylic resin materials for characterization of aesthetic implant restorations.

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## EN

**SYNTHETIC POLYMER TEETH FOR DENTAL PROTHESIS**

- Store in a clean, dry place away from light.
- To ensure optimum bonding to the denture base resin, grind teeth on the cervix prior to setting up, remove wax and degrease with a methyl methacrylate monomer.
- Direct the patient to adopt daily hygiene protocols, suggest him to avoid excessive brushing which could cause abrasive wear, and not to immerse in alkalis, acids or other substances that can damage acrylic materials.
- This product does not contain substances classified as hazardous conforming to Directive 67/548/EEC and subsequent amendments. Consequently, no Material Safety Data Sheet is available.
- Grinding of the teeth generates dust which is classified as "not otherwise classifiable". Do not inhale. Use appropriate protection. TLV/TWA applies.
- Dispose conforming to local law. Product and packing materials are not listed in the Hazardous Waste Inventory as per C.D. 91/689/EEC (14). Patient-contacted materials are at infection risk even for disposal purposes (EU Hazardous Waste Inventory No. 180103).
- Available in the following shade guides: A1-D4.

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Published 2017 - 12 - 15

Quality System Certification:  
ISO 9001 - ISO 13485

ISO 22112:2017  
Type 1-2

MAJOR Prodotti Dentari S.p.A.  
Via Einaudi, 23 - 10024 - Moncalieri (To) - Italy  
Tel +39 011 6400211 - Fax +39 011 6402325  
info@majordental.com  
www.majordental.com

# MAJOR AF COMP Mould Chart

# UPPER and LOWER ANTERIORS

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The anterior teeth include 10 upper and 4 lower moulds, developed according to the most modern aesthetical standards. Available in:

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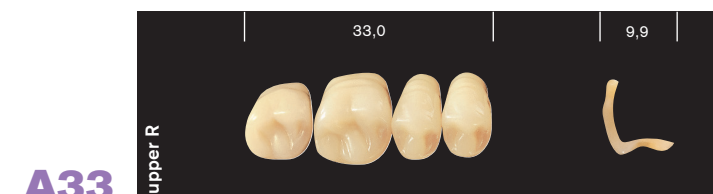
# MAJOR PLUS COMP POSTERIORI

## HIGH ANATOMIC POSTERIORI

High anatomic moulds present an accentuated cuspidal inclination (between 25° and 31°). The morphology of anatomic moulds shows permanently "in touch" surfaces with a consequent decrease of lateral thrust and an increased prosthesis stability. The natural appearance of the tooth, characterized by deep sulcuses, enables a better mastication and give the prosthesis a higher mimetic value.

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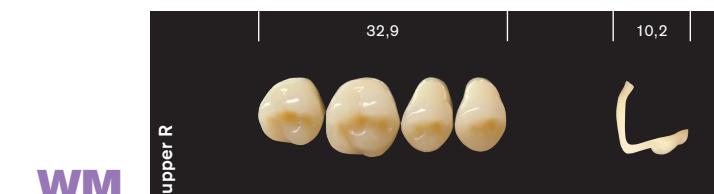
# WFA POSTERIORI

## WFA POSTERIORI FACETS

The new MAJOR AF COMP tooth follows the mounting philosophy according to Prof. Albert Gerber, the palatal cusp of the upper 5th, 6th and 7th tooth occludes in the respective fossa of the lower 5th, 6th and 7th tooth; in the 4th tooth this occlusion is reversed, the vestibular cusp of the lower tooth occludes in the fossa of the upper 4th tooth. The term **WFA (Wide Functional Area)** indicates the type of occlusion done

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
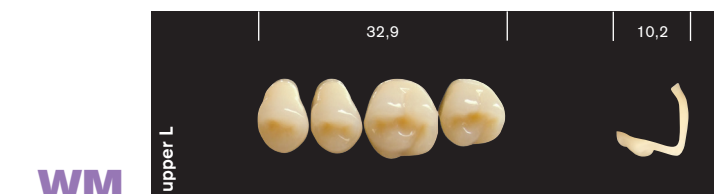
during the mastication, when not only one point of contact is affected but a large functional area of around 2 mm. The occlusal tables respect the concept mortar and pestle. The wide fossa allows a large spectrum of occlusal contacts without interferences. MAJOR AF COMP teeth have such a high precision to reduce to the minimum any possible selective grinding made by the operator.

WM

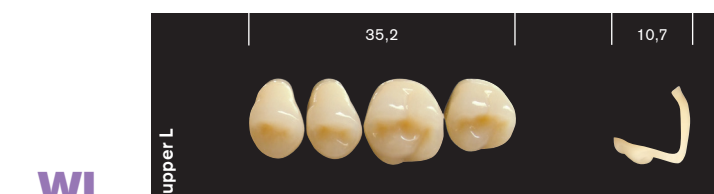WM

WL

WL