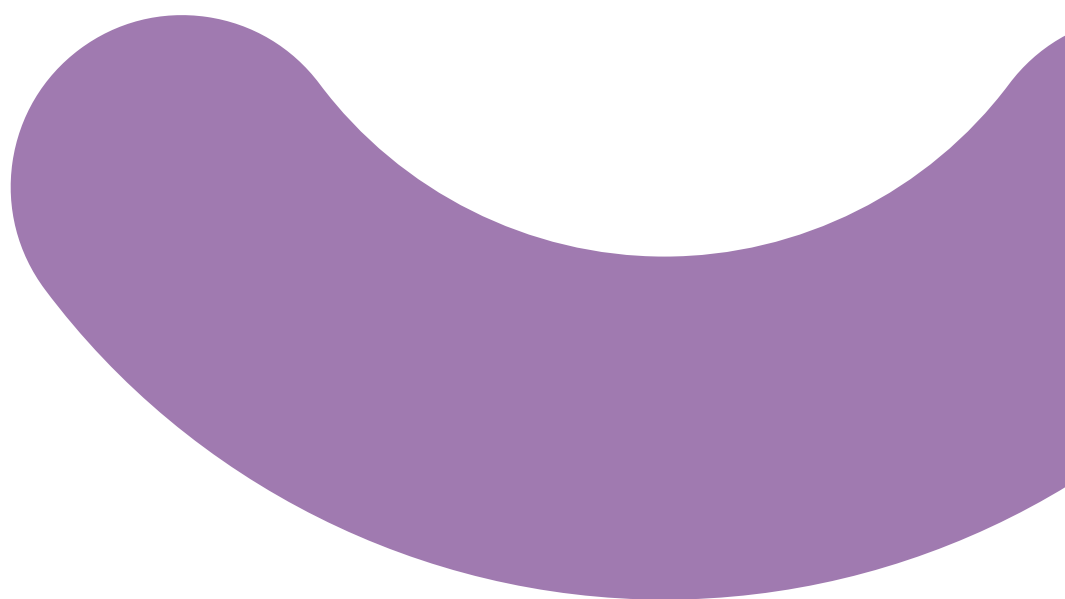
MAJOR AF COMP[®]

MOULD CHART

MAJOR AF COMP

Major AF Comp is a line of multi-layer facings made in composite material specifically developed for the life-like restorations.

Major AF Comp facings may be used for a various range of prosthetic treatments, both by the dentists in clinic and by technicians in laboratory.

Major AF Comp facings are developed to combine with the Major AF Link system, a range of acrylics and light curing composite materials, for the creation and characterization of high aesthetic implant prosthesis.

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UPPER AND LOWER ANTERIORS

ANTERIORS MOULDS

The anterior teeth include 10 upper and 4 lower moulds, developed according to the most modern aesthetical standards. Available in:

- 10 upper shapes in three different sizes: S (small) - M (medium) - B (large)
- 4 lower shapes L

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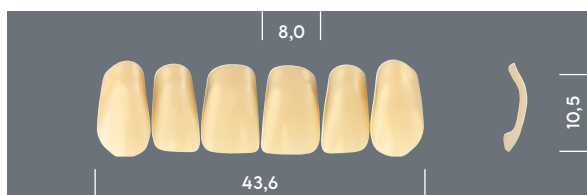
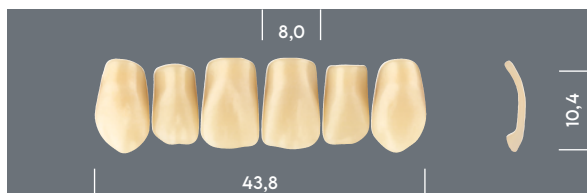
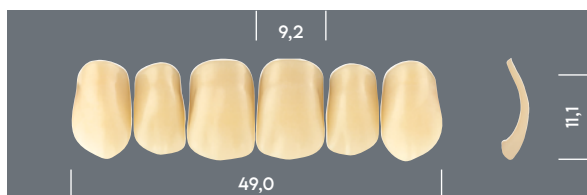
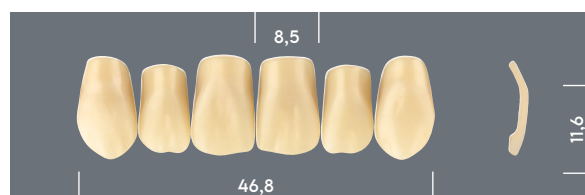
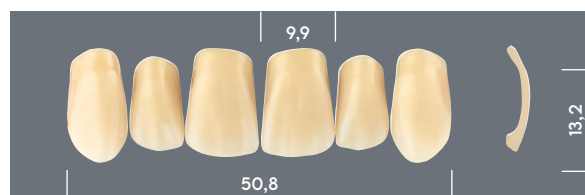
L7

MAJOR AF COMP POSTERIOR

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Anatomic moulds present an accentuated cuspidal inclination. The morphology of anatomic moulds shows permanently "in touch" surfaces with a consequent decrease of lateral thrust and an increased prosthesis stability.

The natural appearance of the tooth, characterized by deep sulcuses, enables a better mastication and give the prosthesis a higher mimetic value.

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TECHNICAL POSTERIOR

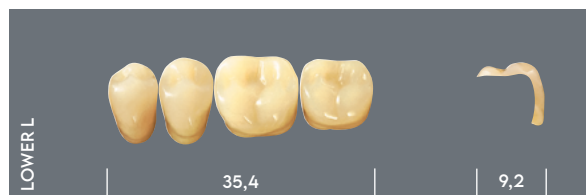
Technical moulds are designed with a balanced occluding plan following specific geometrical rules.

The tooth has a cuspidation with a geometrical inclination superior to 20° and flat sliding plans. This kind of design assures an easy and fast antagonists articulation.

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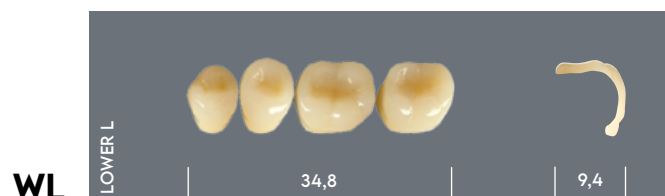
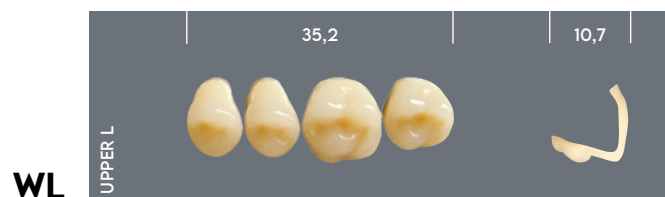
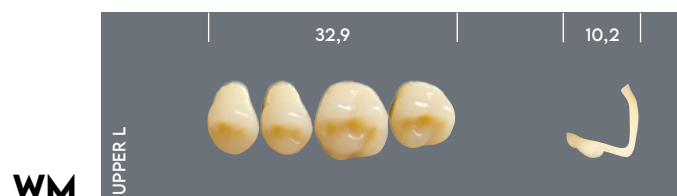
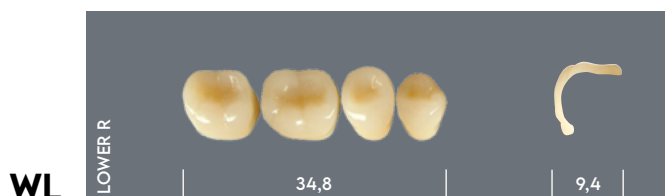
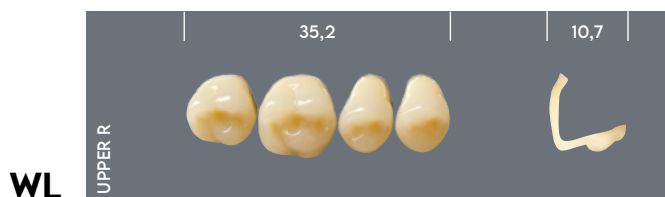
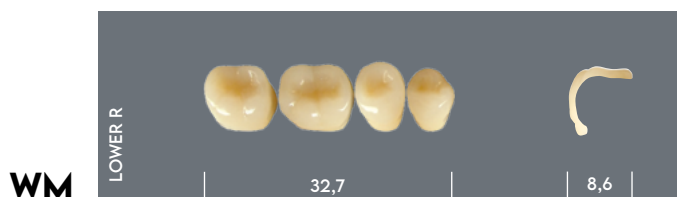
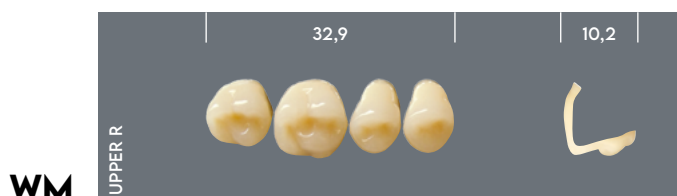
WFA POSTERIORI

WFA POSTERIORI

The new MAJOR AF COMP tooth follows the mounting philosophy according to Prof. Albert Gerber, the palatal cusp of the upper 5th, 6th and 7th tooth occludes in the respective fossa of the lower 5th, 6th and 7th tooth; in the 4th tooth this occlusion is reversed, the vestibular cusp of the lower tooth occludes in the fossa of the upper 4th tooth. The term WFA (Wide Functional Area) indicates the type of occlusion done

during the mastication, when not only one point of contact is affected but a large functional area of around 2 mm. The occlusal tables respect the concept mortar and pestle. The wide fossa allows a large spectrum of occlusal contacts without interferences. MAJOR AF COMP teeth have such an high precision to reduce to the minimum any possible selective grinding made by the operator.

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EN

SYNTHETIC POLYMER TEETH FOR DENTAL PROSTHESIS

- Store in a clean, dry place away from light.
- To ensure optimum bonding to the denture base resin, grind teeth on the cervix prior to setting up, remove wax and degrease with a methyl methacrylate monomer.
- Direct the patient to adopt daily hygiene protocols, suggest him to avoid excessive brushing which could cause abrasive wear, and not to immerse in alkalis, acids or other substances that can damage acrylic materials.
- This product does not contain substances classified as hazardous conforming to Directive 67/548/EEC and subsequent amendments. Consequently, no Material Safety Data Sheet is available.
- Grinding of the teeth generates dust which is classified as "not otherwise classifiable". Do not inhale. Use appropriate protection. TLV/TWA applies.
- Dispose conforming to local law. Product and packing materials are not listed in the Hazardous Waste Inventory as per C.D. 91/689/EEC 1(4). Patient-contacted materials are at infection risk even for disposal purposes (EU Hazardous Waste Inventory No. 180103).
- Available in the following shade guides: A1-D4.

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MAJOR Prodotti Dentari S.p.A.